

SomnoDent™ MAS Acrylic

Cleaning & Storage Instructions

An Acrylic MAS is identified by a number of metal ball clasps on the inner (tooth side) surface of the appliance. The appliance is made of one material – the high quality acrylic.

1. Rinse in cold or lukewarm water and then clean with a soft toothbrush and a mild detergent, such as liquid soap. Then rinse well.

Pay particular attention to cleaning the metal components and the fitting surface to ensure that it is free from all calculus or other scaling deposits. Avoid skin contact with the metal components where possible.

2. Store in a glass of clean water.

By storing in water, the acrylic will not dry out and will prevent crack corrosion and possible discolouration of the metal components (caused by build up of salts and saliva).

3. Once or twice a week soak in a partial denture cleaner, such as “CALDENT”, “STERADENT - ACTIVE PLUS” OR “FRESH ACTIVE POLIDENT”.



Tablets are available from chemists and most supermarkets. Soak for 5 minutes only, then rinse and store in water, as above. Always follow the instructions on the pack.

Your SomnoMed device has metal components that may be damaged by some denture cleaners. Ensure that you use a partial denture cleaner.

For more information:
www.somnomed.com.au
or call 1800 445 660



SomnoMed®